

PlusRA Diagnostic Tool



BodyBox

To go into Diagnostics by clicking on the green Oscilloscope icon, then select **Devices**, and **Sdasa-0**, If you have more than one device or it doesn't connect, select **Sdasa-1**. You will now have communication with your device.

Select **Acquisition**, and then the device you wish to use (**Bodybox BBx / GEM**). In the example below, we used a **BodyBox**.

Sample BBX will bring up the BodyBox channels. If you now go into **Control**, and select, **Set BBx Cal**, This will start the BodyBox calibration. You can select **Set BBx Test**, this will open and close the Vent valve. **Set BBx Door** operates the Door magnets to check the door function and the rubber seal. Once completed, select **Control**, and select **SetALLOFF**

If you select **Tools**, and **Auto Null Box**, this re-sets the Zero .

If any Pots are adjusted in the Interface, goto **Tools** select **Zero DAC**

Exit the Diagnostic program, and perform all BodyBox calibrations from the normal Calibration screen.

