

Cleaning the Source

Cleaning of the source, This procedure will take around an hour to perform, plus removing the source from the housing And refitting.

During this process we use KoKo's Zapet and lens cleaning alcohol wipes.

The source should be held upside down and with a cotton bud dipped in Zapet, then wipe around the reflector until cotton bud needs changing, repeat until the source is clean.

Now wrap an alcohol wipe around cotton bud and clean the reflector.

As you can see from pictures, this cleans the source to a reasonable standard, it can bring the CH4 gain down from as high as 1.23 to around 97. This process generally will last around for 4 -6 weeks before either the source is changed or cleaned again. Fig 1. Before Fig 2. After

