


Creating a New Database

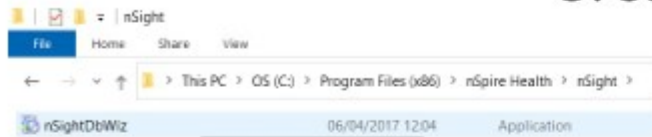

Select path above, and double click on: **nSightDbWiz**

In the **Server Connection Settings** window enter the name or IP address of the database server. If the database resides on the same machine as this install then enter **(local)** into the **Server** field. Uncheck the box stating **Use Windows Integrated Authentication**. Enter the Microsoft SQL Server administration account, **sa**, into the **User ID** field. Enter the **sa** account's password in the **Password** field, click **Next**.




Select Database Name.



Confirm Database Name. YES



Click on OK

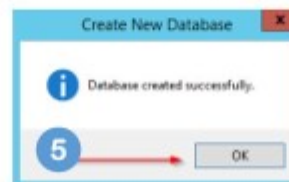

Click on Save Log & Select FINISH

