

Main (Part 1) Service Overview - PX3000 / PX4000

Before any service is performed, a full system calibration is performed to insure the system is functional.

1. **SWITCH THE UNIT ON**, and allow it to warm up (30 - 45 minutes).

From the main calibration screen, you will have to perform two calibrations, one for the **GEM**, and one for the **BodyBox**.

After both sets of calibrations have been performed, you can proceed with the PX4000 Part 1 Service, changing the parts below

System Calibration

Instruments					
Instrument	Serial Number	Technology	Calibration Date	Site	Workstation
Plethysmograph	I215002		9/20/2017 5:30 PM	HOS	MININT-CUGAHQ0
GEM	IE16027		9/20/2017 5:51 PM	HOS	MININT-CUGAHQ0
Plethysmograph	BPD1099		3/29/2016 9:13 AM	X Default Site	NSIGHTCOMPUTER




O2 Cell



O2 Cell Fitted



Balloon Assemblies

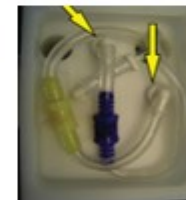

GEM "O" Rings



Valve Housing



One Way Valve



Table 1 - HD/Px 3000 HD/Px 4000-N2 PM Kit (Part Number K710081)

Content	Quantity
GEM valve blue balloon assembly	1
GEM valve yellow balloon assembly	1
O2 analyzer	1
One-way exhalation valve	1
Nafion tube	1
Nafion filter	1
GEM valve O-rings	3