

Restoring a Database



After creating a new blank database...

To restore a database go to: C: / Program Files (x86) / nSpire Health / nSight (As above) & select nSightBackup .

This will open a window as in Fig 1. Select your NEW Database, tick on Force Overwrite, and select RESTORE.

A box will open as in Fig 2. Select your OLD Database name .Bak from a location / flash drive, and select OPEN.

The program will ask twice are you sure you want to overwrite the NEW Database, select YES to both.

The program will restore the .Bak file into the NEW Database. After some time a message will come up Restore Complete.

