


Connection Trouble Shooting 2

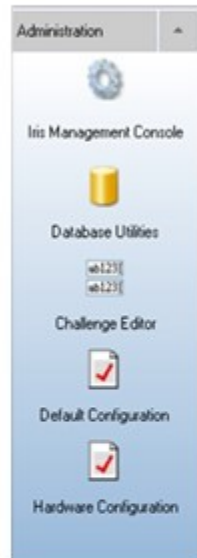

From the main KoKo front page, Select Administration, Default Configuration. A box will appear as in Fig 1. Check the Application Path: Binary Path: Config Path: Resource Path: & Calibration Path: As in Fig 1. Now select the Forced Vital Capacity tab. Double click on Product. This should bring up Spirometer and serial number.

Double click on Testing Device, This should bring up the device used for measurement.

For SX1000 (Spirometer).

Now go into Hardware Configuration, A box will appear as in Fig 2. Select the Spirometer tab, double click on Interface this should bring up Spirometer and serial number. Now double click on Pneumotach Type, This should be Spirometer for SX1000.

Perform a full calibration and Spirometry test to confirm the system is working.

